# Miru

## 1day Menicon Flat Pack

### PATIENT INSTRUCTIONS

#### Miru 1day Menicon Flat Pack (hioxifilcon A) Daily Disposable Soft Contact Lens

**CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED PRACTITIONER.**

#### Table of Contents

Congratulations  
Introduction  
Wearing Restrictions and Indications  
How Supplied and Opened  
Contraindications (Reasons not to use)  
Warnings  
Precautions  
Adverse Reactions  
Personal Cleanliness and Lens Handling  
Caring for your Lens  
Emergencies  
When to call your Practitioner  
Instructions for Monovision Wearer  
Wearing Schedule  
Appointment Schedule

#### CONGRATULATIONS:

You have just received your new **Miru 1day Menicon Flat Pack (hioxifilcon A)** Daily Disposable Soft Contact Lens. This booklet has been prepared to help you care for it. Please read it carefully and follow the instructions so that you receive full satisfaction from your lens.

PRACTITIONER: \_\_\_\_\_

ADDRESS: \_\_\_\_\_

TELEPHONE: \_\_\_\_\_

**Lubricant/Rewetting Drops:** \_\_\_\_\_

**Rewetting Solution:** \_\_\_\_\_

#### INTRODUCTION:

With your decision to wear soft (hydrophilic) contact lens, you have joined a growing number of people who are discovering new pleasures from this important advance in vision correction.

Your **Miru 1day Menicon Flat Pack** is made of a stable, inert, highly purified, hydrophilic (water absorbing) polymer. Fully molded to optical precision in the dry state, your lens holds these qualities in the soft, moist state in which they are suited for your use. Kept moist by the tears in your eyes, the lens is soft and comfort can be immediate. You are cautioned, however, to follow the initial wearing time schedule prescribed by your practitioner and not to overwear the lens simply because they remain comfortable. Your eye care practitioner will determine your appropriate wearing schedule.

Although soft and pliable, your lens is strong and durable. **Miru 1day Menicon Flat Pack** is designed as a Daily Disposable and should be worn one day only. As with all precision devices, proper use will assure you the benefits of convenience, comfort, and confidence in your lens.

Read this Patient Instructions carefully. It contains the information you need to know to wear, handle, and care for your **Miru 1day Menicon Flat Pack**. If you are in doubt about any instructions, request clarification from your eye care practitioner.

#### WEARING RESTRICTIONS and INDICATIONS:

The **Miru 1day Menicon Flat Pack** is indicated for daily wear single use only for the optical correction of refractive ametropia (myopia and hyperopia) in phakic persons with non-diseased eyes who may have 1.00D or less of astigmatism.

The lens is intended to be worn once and then discarded at


the end of each wearing period on a daily basis. The patient should be instructed to start the next wearing period with a new lens.

The **Miru 1day Menicon Flat Pack** described in this booklet should be designed as a Daily Disposable.

The lens is designed for daily wear (less than 24 hours while awake). The maximum wearing time should be determined by the eye care professional based upon your physiological eye condition because individual responses to contact lenses vary. Patients tend to overwear the lenses initially. The eye care professional should stress the importance of adhering to the initial maximum wearing schedule. Studies have not been conducted to show that **Miru 1day Menicon Flat Pack** is safe to wear during sleep; therefore remove your lenses while sleeping. Normal daily wear of lenses assumes a minimum of 6 hours non-lens wear per 24 hour period. Optimum individual wearing schedule will vary.

#### HOW SUPPLIED and OPENED

Each lens is supplied sterile, in a non-traditional packaging system. The Flat Pack contains buffered saline solution. Each container is marked with base curve, dioptric power, diameter, Single Patient Use, Rx Symbol, Sterile Symbol, composition of the lens, manufacturing lot number and expiration date of the lens.

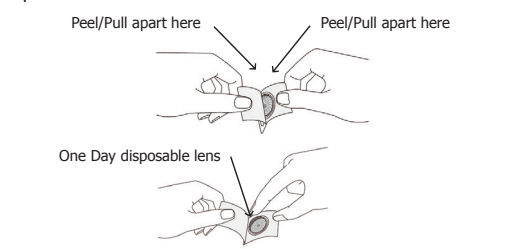To open the Flat Pack, grasp both top and bottom foil tabs and peel them apart to fully expose the lens.

#### DO NOT WEAR YOUR Miru 1day Menicon Flat Pack WHILE SLEEPING.

#### CONTRAINDICATIONS (REASONS NOT TO USE): DO NOT USE the Miru 1day Menicon Flat Pack when any of the following conditions exist:

- Acute and subacute inflammation or infection of the anterior chamber of the eye.
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids.
- Severe insufficiency of lacrimal secretion (dry eyes).
- Corneal hypoesthesia (reduced corneal sensitivity), if not-aphakic.
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Any active corneal infection (bacterial, fungi, or viral)
- If eyes become red or irritated.
- Patients unable to follow lens wearing schedule or unable to obtain assistance to do so.

#### WARNINGS:

- Problems with contact lenses could result in serious injury to the eye. It is essential that you follow your eye care practitioner's direction and all labeling instructions for proper use of lenses. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision; therefore, if you experience eye discomfort, excessive tearing, vision changes, or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner.
- Daily wear lenses are not indicated for overnight wear, and you are instructed not to wear the **Miru 1day Menicon Flat Pack** while sleeping. Clinical studies have shown that the risk of serious adverse reactions is increased when these lenses are worn overnight.
- Studies have shown that contact lens wearers who are smokers have a higher incidence of adverse reactions than nonsmokers do.

#### PRECAUTIONS:

- The lens is intended to be worn once and then discarded at the end of each wearing period.
- You should be instructed to start the next wearing period with a new lens.
- Emergency lens cleaning and disinfection is not recommended.
- You should be reminded to have replacement lenses or back-up spectacles available at all times.
- Always wash and rinse hands before wearing lenses.
- If the lens sticks (stop moving) on the eye, follow

- the recommended directions on Care for a Sticking (non-moving) Lens. The lens should move freely on the eye for the continued health of the eye. If non-movement of the lens continues, you should IMMEDIATELY consult your eye care practitioner.
- Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in your eyes or on the lenses.
- It is best to put on lenses before putting on makeup. Water-base cosmetics are less likely to damage lenses than oil-base products.
- Do not touch contact lenses with your fingers or hands if the hands are not free of foreign materials, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury to your eye.
- Carefully follow the handling, insertion, removal, and wearing instructions for the **Miru 1day Menicon Flat Pack** and those prescribed by your eye care practitioner.
- Never wear lenses beyond the period recommended by your eye care practitioner.
- If aerosol products such as hair spray are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.
- Always handle lenses carefully and avoid dropping them.
- Avoid all harmful or irritating vapors and fumes while wearing lenses.
- Ask your eye care practitioner about wearing lenses during sporting activities.
- Inform your health care practitioner about being a contact lens wearer.
- Never use tweezers or other tools to remove the lens from the lens container unless specifically indicated for that use.
- Do not touch the lens with fingernails.
- Always contact your eye care practitioner before using any medicine or medications in the eyes.
- Always inform the employer of being a contact lens wearer. Some jobs may require use of eye protection equipment or may require that you not wear contact lenses.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of your eyes. You should be instructed as to a recommended follow-up schedule.

#### ADVERSE REACTIONS:

- The following problems may occur:
- Eyes stinging, burning, itching (irritation), or other eye pain.
  - Comfort is less than when lens was first placed on eye.
  - Feeling that something is in the eye such as a foreign body or scratched area.
  - Excessive watering (tearing) of the eye.
  - Unusual eye secretions.
  - Redness of the eye.
  - Reduced sharpness of vision (poor visual acuity).
  - Blurred vision, rainbows, or halos around objects.
  - Sensitivity to light (photophobia).
  - Dry eyes.

#### If you notice any of the above, IMMEDIATELY REMOVE YOUR LENS.

- If the discomfort or problem stops, then look closely at the lens. If the lens is in any way damaged, **DO NOT PUT THE LENS BACK ON YOUR EYE.** Discard the lens and replace with a fresh lens.
- After reinsertion, if the problem continues, you should **IMMEDIATELY REMOVE THE LENS AND CONSULT YOUR EYE CARE PRACTITIONER.**

When any of the above problems occur, a serious condition such as infection, corneal ulcer, neovascularization, or iritis may be present. KEEP THE LENS OFF THE EYE AND SEEK IMMEDIATE PROFESSIONAL IDENTIFICATION of the problem and prompt treatment to avoid serious eye damage.

#### PERSONAL CLEANLINESS and LENS HANDLING: Before Wearing Your Lens:

- Cleanliness is an important aspect of contact lens care.**
- Before wearing your lenses, always wash and rinse your hands thoroughly and dry them with a lint-free towel.
  - Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in the eyes or on the lenses.
  - Take care in handling your lenses. Always avoid touching your lens with your fingernails or other sharp objects.
  - NEVER WORK DIRECTLY OVER A SINK WITH THE DRAIN OPEN, AS THE LENS MAY BE LOST.

#### Handling and Placing the Lens on the Eye:

1. To avoid the possibility of lens mix-ups, always start

2. Place the lens on the tip of the index finger of your dominant hand.
3. While positioned on your index finger, check to ensure the lens has not turned inside out. To check this, look at the profile of the lens against a light background. If the edge profile appears convex and bowl-shaped, then it is correct. If the lens is inverted, it will flare out at the edge. If the lens is inverted, simply reverse it by using light fingertip pressure. Be sure to avoid damaging the lens with your fingernails.
4. Look straight ahead and raise the upper lid with your other index finger.
5. Then look down, keep both eyes open and place the lens on the upper white part of the eye.
6. Slowly release upper lid, and gently close your eye. The lens should center automatically, or it can be moved on center by gentle fingertip pressure through the lids.
8. Repeat the above procedure for the second lens.
9. If the lens does not move freely on your eye, contact your eye care practitioner for further instructions. There is no single "right way" of putting on lens. If you find this method of lens placement difficult, your eye care practitioner will suggest another method or provide additional information.

NOTE: If after placement of the lens your vision is blurred, check for the following:

- The lens is not centered on the eye (see 'Centering the Lens' next in this booklet).
- If the lens is centered, remove the lens (see 'Removing the Lens' section) and check the following:
  - a. Cosmetic or oils on the lens. Discard lens and place a new lens on the eye again.
  - b. The lens is on the wrong eye.
  - c. The lens is inside-out (it would also not be as comfortable as normal).

If you find that your vision is still blurred after checking the above possibilities, remove both lenses and consult your eye care practitioner.

#### Centering the Lens:

Very rarely, a lens that is on the cornea will be displaced onto the white part of the eye during lens wear. This can also occur during placement and removal of the lenses if the correct techniques are not performed properly. To center a lens, follow the procedure outlined below.

Using your index finger, gently apply pressure to the lens and slide it back on to the cornea. If the lens gets under the upper lid, gently massage the upper lid while looking down and move the lens toward the cornea.

#### Removing the Lens from the Eye: Preparation:

1. Wash and rinse your hands thoroughly.
2. Dry hands with a lint-free towel.
3. Check that the lens is centered on the cornea before attempting to remove the lens. Check your vision by covering one eye. If vision is blurry, the lens is off-center. Re-center the lens before attempting to remove it.

#### Removal:

1. To avoid the possibility of lens mix-ups, always begin with the same lens.
2. Look up and keep both eyes open.
3. Using the middle finger of your dominant hand, gently pull down the lower lid of the first eye. Using the tip of your index finger of the same hand, touch the lens and slide it onto the white of the eye.
4. Gently "pinch" the lens between the index finger and the thumb and remove.
5. Repeat the procedure for the second eye.
6. If the lens still cannot be moved, contact your eye care practitioner for further instruction.
7. Upon removal, discard the lens.

**IMPORTANT:** Always avoid touching your lens with your fingernails. Use only your fingertips.

If you find this method difficult, your eye care practitioner will suggest another method or provide additional instruction.

#### CARING FOR YOUR LENSES: Basic Instructions:

The **Miru 1day Menicon Flat Pack** described in this booklet is designed as a Daily Disposable.

The lens is intended to be worn once and then discarded

at the end of each wearing period. You are instructed to start the next wearing period with a new lens. You are also instructed to always have a spare pair of lenses with you at all times.

The lens is designed for daily wear (less than 24 hours while awake). The maximum wearing time should be determined by your eye care professional based upon your physiological eye condition because individual responses to contact lenses vary. Patients tend to overwear the lenses initially. Your eye care professional should stress the importance of adhering to the initial maximum wearing schedule. Studies have not been conducted to show that **Miru 1day Menicon Flat Pack** is safe to wear during sleep; therefore remove your lenses while sleeping. Normal daily wear of lenses assumes a minimum of 6 hours non-lens wear per 24 hour period. Optimum individual wearing schedule will vary.

You should adhere to a recommended daily disposable care regimen. Failure to follow the regimen may result in development of serious ocular complications as discussed in the warning section above.

If you require only vision correction, but will not or cannot adhere to a recommended care regimen for your lenses, or are unable to place and remove lenses or have someone available to place and remove them, you should not attempt to get and wear contact lenses.

When you first get your lenses, be sure you have to put the lenses on and remove them while you are in your eye care practitioner's office.

Your eye care practitioner should instruct you about appropriate and adequate procedures and products for your use, and provide you with a copy of the Patient Instructions for the **Miru 1day Menicon Flat Pack**.

For safe contact lens wear, you should know and always practice your lens care routine:

- Always wash, rinse, and dry hands before wearing contact lenses.
- Do not use saliva for lubricating or rewetting your lenses. Do not put lenses in the mouth.
- Never rinse your lenses in water from the tap. There are two reasons for this:
  1. Tap water contains many impurities that can contaminate or damage your lenses and may lead to eye infection or injury.
  2. You might lose the lens down the drain.
- Your eye care practitioner may recommend a lubricating/rewetting solution for your use. Lubricating/Rewetting solutions can be used to wet (lubricate) your lenses while you are wearing them to make them more comfortable.

#### Care for a Sticking (non-moving) Lens:

If the lens sticks (cannot be removed), you should apply 3 to 4 drops of the recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after 15 minutes, you should IMMEDIATELY consult your eye care practitioner.

#### EMERGENCIES:

If any chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, you should:

#### FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT YOUR EYE CARE PRACTITIONER OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.

#### WHEN TO CALL YOUR PRACTITIONER?

Certain symptoms may be early indicators of potentially serious problems. A careful examination of your lens, and professional examination of your eyes, may be required. Remove the lens following the instructions outlined in this guide, and call your eye care practitioner if:

1. Your eye becomes red and feels irritated or "gritty".
2. You notice a change in your vision or see rainbows or halos around objects.
3. You experience discomfort and/or sensitivity to lights.

#### A good general policy is: "IF IN DOUBT ... TAKE THE LENS OUT" and contact your eye care practitioner.

#### Learn and Use Proper lens Care Habits:

1. Follow instructions.
2. Handle lens properly.
3. Learn how to put on and take off your lens.
4. Always use a fresh lens at the start of the wearing period.

#### INSTRUCTIONS FOR MONOVISION WEARER

You should be aware that as with any type of lens correction, there are advantages and compromises to monovision contact lens therapy. The benefit of clear near vision in straight ahead and upward gaze that is available with monovision may be accompanied by a vision compromise that may reduce your visual acuity and depth perception for distance and near tasks. Some patients have experienced difficulty adapting to it. Symptoms, such as mild blurred vision, dizziness, headaches and a feeling of slight imbalance, may last for a brief minute or for several weeks as adaptation takes place. The longer these symptoms persist, the poorer your prognosis for successful adaptation. You should avoid visually demanding situations during the initial adaptation period. It is recommended that you first wear these contact lenses in familiar situations, which are not visually demanding. For example, it might be better to be a passenger rather than a driver of an automobile during the first few days of lens wear. It is recommended that you only drive with monovision correction if you pass your state driver's license requirements with monovision correction. Some monovision patients will never be fully comfortable functioning under low levels of illumination, such as driving at night. If this happens, you may want to discuss with your eye care practitioner having additional contact lenses prescribed so that both eyes are corrected for distance when sharp distance binocular vision is required. If you require very sharp near vision during prolonged close work, you may want to have additional contact lenses prescribed so that both eyes are corrected for near when sharp near binocular vision is required. Some monovision patients require supplemental spectacles to wear over the monovision correction to provide the clearest vision for critical tasks. You should discuss this with your eye care practitioner.

It is important that you follow your eye care practitioner's suggestions for adaptation to monovision contact lens therapy. You should discuss any concerns that you may have during and after the adaptation period.

The decision to be fit with monovision correction is most appropriately left to the eye care practitioner in conjunction with you, after carefully considering and discussing your needs.

**WEARING SCHEDULE:** THE WEARING AND REPLACEMENT SCHEDULE SHOULD BE DETERMINED BY YOUR EYE CARE PRACTITIONER.

The **Miru 1day Menicon Flat Pack** is indicated for daily wear. The maximum suggested wearing time for these lenses are:

DAY	1	2	3	4	5	6 and after
HOURL	6	8	10	12	14	All waking hours

#### STUDIES HAVE NOT BEEN CONDUCTED TO SHOW THAT THE "Miru 1day Menicon Flat Pack" IS SAFE TO WEAR DURING SLEEP.

#### APPOINTMENT SCHEDULE:

Regular check-up examinations by your eye care professional are an important part of wearing contact lenses. It is recommended that you follow your eye care professional's directions for follow up examinations. Keep all appointments for your check-up visits. If you move to a new city, ask your present eye care professional to refer you to an eye care professional in your new location.

#### IMPORTANT:

In the event that you experience any difficulty wearing your lenses or you do not understand the instructions given to you, DO NOT WAIT for your next appointment. TELEPHONE YOUR EYE CARE PRACTITIONER IMMEDIATELY.

**Menicon America, Inc.**  
**Waltham, MA 02451**  
**1-800-MENICON (1-800-636-4266)**  
**information@menicon.com**

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